(((

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000

APP functions and setting

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Personal information and exercise goals After entering into APP, please first set up personal information. Me _Click avatar

· Head portrait, gender, age, height and weight can be set up and personal information can enhance accuracy of data. Customize personal everyday exercise goals. A reasonable exercise plan is helpful for promoting body quality.

Heart rate monitor blood

pressure Fitness Tracker

User Guide

Wireless connection: after successfully matching, APP wil

address. When APP is opened or operated in background, it

Data Synchronism: manually pull down the synchronization data on the APP home page; the bracelet can store off-line data for 7 days, the more data, the

longer the synchronization time is, and the longest time is

automatically save the wristband Wireless connection

will automatically search and link with the wristband

ender	Male >
ge	20 >
eight	170Cm >
leight	60Kg >
nit	Metric system >

Part of introduction

I Charging method

Wearing method

0000000000

Adjustment Hole Display screen Wrist strap

first pace, start the unit tocheck whether there is power supply, Power on not allowed: please use it it after it is fully charged. When the wristband is

nark, and then insert the USB plug into

thewristbandAfter the wristband is unpacked for the

charged, pull off the watchbelt with the flashing

hecomputer USB port or charging end for charging.

Interface to automatically measure the current oxyger

Message reminding: when a text message arrives, the

displays the name, and the short message number of the OO reminding: when the OO information arrives, the ring

wristband will vibrate and display the SMS icon reminder; the

vill vibrate and display the QQ icon.

address message of the address book contact person

2. Adjust the size suitable for the size of wrists via

nent Hole, fasten wrist strap button

1.In the state of the wristband shutdown, Charging or

long press the function button for 3 seconds to vibrate the wristband;

2.When the wristband is in the boot state, Switch to the

shutdown interface, long press the function button for 3

1. When the device starts up, to touch function keys for a

display; 2.The screen shall be turned off without operation and

3.Wristband factory default to open calls and SMS reminders. Users can choose to open or close in the main interface of the APP/Android.

short time period can lighten screen or switch over

mobile phone APP to revise off-screen time

3. Sensor shall be close to skin avoiding moving

WeChat reminding: when the WeChat information is received, the hand ring will shake and the WeChat icon reminds

Alarm clock reminding: it can set up three alarm clocks, and all synchronize to the bracelet after setting up. The intelligent alarm clock is a silent alarm clock. When the alarm is on, the bracelet will cause moderate vibration, and support the offline alarm clock reminder.

Other reminding: when receiving email or other information, the wristband vibrates and displays other reminder icons. Click to see the content. vibrate and display other reminder icons, Click the viewable content.

WeChat reminding: When the steps reach the target set on the APP, the wristband will vibrate and display the targe

Sleep detection function

When you fall asleep, the wristband will automatically determine the sleep monitoring mode, automatically detect your deep sleep / shallow sleep / wakefulness time, calculate your sleep quality, and sleep data temporarily support APP

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1.Best wear the wristband behind malleolus ulnaris;

For promoting function, a promoting switch shall be switched on in APP-end Setting Center, mobile phone and wristband shall be kept to be connected successfully and there will be vibrating reminders when there are mes

Call reminding: when the caller calls, the ring will shake and the name or number of the caller is displayed. (if the caller's name has been saved in the phone address book, the name is displayed, otherwise the display number)

movement, return to the specific motion interface.

concentration in human blood. You can also connect to APP for testing at the oxygen-blood interface. Message Interface: Blood Pressure Interface: Switch to the Blood Pressure Interface to automatically test the current diastolic and systolic blood pressure of the huma oody. You can also test the blood pressure interface

I Training interface

Switch to the interface, long press three seconds into the specific sports interface, including rope skipping, cycling. Runing and other three sports, click touch to switch the specific sports interface.

A long three seconds return can exit the specific

hree seconds long specific motion interface can start corresponding movement, into the motion details interface, long press three seconds to end the current

In the message interface, press three seconds to view the message, click on the page, view the information can exit or press three seconds to exit.

Functional interface

You can enter the lower menu interface with more than three seconds and other functions, and perform more

Return: The interface can be pulled out for three seconds, and the interface can be returned to the main interface

of the wristband.

• Please do not disassemble, repair or transform the machine on its own.

 \bullet The use of the environment is 0 degrees $\sim\!45$ degrees, and it is forbidden to hrow it into the fire so as not to cause

Equipment type	0.96 inch smart wristband	Battery type	Lithium polymer
Display screen	0.96 inch IPS color screen	Battery model	341423
Product weight	22g	Battery capacity	90mAh
Full length of wristband	23.0cm	Input power	DC 5.0V
Waterproof level	IP68	power of battery	0.333Wh

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the charging operation with a soft cloth, otherwise it will cause corrosion of charging contact point, charging fault. • Do not touch the chemical substances such as gasoline,

· Please do not use this product in high pressure and high magnetic environment • If you have sensitive skin skin or tighten the wristband,

clean solvent, propanol, alcohol or insect repellent

you may feel discomfort.

Step gauge interface: Display the number of steps on

Heart Rate Interface: Switch to Heart Rate Interface

and automatically test the current heart rate. You can

. Please wipe the water, the wristband can be used only for

Distance Interface: Show the distance of the day

Calories Interface: Displays the current energy

Please dry the sweat drips on the wrist in time. The strap has long contact with soap, sweat, allergies or pollution ingredients, which may cause skin allergy itching. • If it is often used, it is recommended to clean the wristband every week. Wipe with wet cloth and remove oil or dust with mild soap.

• It is not appropriate to wear a hot bath with a wristband. After swimming, please wipe the wristband in time so as to keep dry.

Why is the first test of heart rate too long? To test your heart rate, you need to take a long time signal

to accurately calculate your heart rate What do Wireless connections do? (connection reconnection failure or connection speed is slo Confirm whether your device is 4.4 or more versions of IOS 8.0 and Android system and support Wireless connection

- a. Because of the problem of signal interference in wireless connection, there may be no time to connect each other. If the connection is not long enough, please ensure that the operation is carried out without magnetic field or without multi Wireless connection device interference.
- c. Turn off the mobile phone background application or restart the mobile phone d. Mobile phones do not connect to other Wireless connection devices or functions at the same time.

e. APP is running normally in the background and may not be connected if it is not in the background. Can't search the wristband?

b. Turn off Wireless connection and open

Since the wristband is not searched by phone in Wireless connection broadcast, make sure the wristband is in power and activated and is not connected to any other phones. Then, put the wristband close to the phone, if it is still invalid, turn off the phone, and after about 20 seconds, restart the phone Wireless connection

Why the wristband should be worn more tightly when

The wristband uses light reflection principle, penetrates the skin through the light source, and collects the signal reflected to the sensor, to calculate your heart rate. If you don't wear it tightly, there will be ambient light reaching into the sensor,

which will affect measurement accuracy, Why not remind when opening the reminder function?

with not remind when opening the reminder function?

Android phone: confirm that the phone and the wristband are connected, and open the corresponding permissions in the phone settings, allowing the "HagFitt20" APP access calls, messages and address book, and keeping "HagFitt20" running in the background; if the phone is installed with security software, add "GloryFit" to "trust".

Iphone: If there is no reminder after the connection, it is recommended to restart the phone to reconnect, when reconnecting the wristband again, there will be reminder after popping up * Wireless connection pairing request* and clicking "Pairing". Is the wristband waterproof?


It supports IP67 waterproof and dust-proof level (IP67 standard is 35 degrees below water, 1 meters below 30 minutes). Usually, wash hands rain, cold water shower, wash car can be worn with heart. Does the Wireless connection need to be open all the

time? Is there any data after disconnection

Before the data is synchronized, the data is kept on the wristband host (seven days of data can be retained).
When the Wireless connection of the wristband and the wristband is successful, the wristband data will be automatically uploaded to the mobile terminal. Please vnchronize the data to the mobile terminal in time

Note: if the caller and SMS alerts are opened, you must keep the Wireless connection, and the alarm is offline.

Equipment type	0.96 inch smart wristband	Battery type	Lithium polymer
Display screen	0.96 inch IPS color screen	Battery model	341423
Product weight	22g	Battery capacity	90mAh
Full length of wristband	23.0cm	Input power	DC 5.0V
Waterproof level	IP68	power of battery	0.333Wh
Wrist strap	TPU watch	Packing	Host,
material	band	List	manual

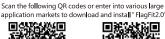Device requirement: iOS8.0 and above; Android 4.4 and above to support Wireless connection

When being used for the first time, it is necessary to write being used to the installant, its little-cessary to connect APP for alignment. After successful connection, wristband will self-synchronize time, or otherwise, step calculation and sleep data won't be accurate.

Open mobile phone APP Click to set up icons

Add device (iOS)/ Add device(Android)

Click the upper right to find the icon Click device attachment

Note: you can get sleep data when you go to sleep with a hand

Main interface

The wristband has three main interfaces, and the interface can be switched after three seconds at the main interface. After the first pairing of APP the wristband will synchronize the time, date

Find the phone: In the connection state, long Click to find the mobile phone interface, the phone will ring out to mware version number and Wireless connection address

Restore settings: Press three seconds on the interface, and all the ring data will be cleared, and the APP data will be cleared that day (the rest of the historical data will remain). Shutdown: The interface can be shut down for three seconds and the wristband will vibrate.

•Please avoid strong impact, extreme heat and exposure to the wristbandt.

 \bullet This machine is charged with 5V 500MA, and it is strictly forbidden to use overvoltage load power.