# ITSU

Rev.1.0

User Manual  
IS4028使用手冊

# CONTENTS

Safety Instructions	2
Warranty Claims	4
Product Parts	5
Moving the Chair	6
Controller Diagram	7
Product Operation Guide	9
Product Features	24
Cleaning And Maintenance	25
Questions And Answers(Q&A)	26
Troubleshooting	27
Product Specification	29
Contact Us	31

# Safety Instructions

## Warning!

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in the risk of electric shock.

Check with a qualified electrician and service personnel if you are in doubt as to whether the product is properly grounded. Do not modify the product's accompanying plug. If the outlet is not suitable, be sure to have a proper outlet installed by a qualified electrician.

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire, or injury.

Read the following details before use:

1. Be sure to insert the massage chair plug into the proper electrical outlet.
2. Make sure that the power plug is securely inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use and before cleaning.
4. Do not use the massage chair in places where the humidity is high.
5. Do not use the massage chair outdoors and avoid places with direct sunlight.
6. If the power goes off during use, be sure to turn the power switch to the OFF position and pull the power plug out from the socket to prevent any damages when the power comes back.
7. Never operate the massage chair if the power cord is wet or damaged.
8. Never plug/unplug the power cord from the socket with wet hands.
9. Close and continuous supervision is necessary when this appliance is used by or near children, invalids or disabled persons.
10. Use this appliance only as described in this user manual.
11. Do not use any attachments other than those recommended by ITSU.
12. Do not stand, sit or jump in the calves rest when the massage chair is in use. This may result in an accident or injury.
13. Do not attempt to stand on the backrest and armrest .This may result in an accident or injury.
14. Be careful not to fall asleep during the massage.

# Safety Instructions

15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair.
16. Do not remove the massage chair by pulling the power supply cord.
17. Be sure to check that there is no one (children in particular), pets or obstacles under the massage chair prior to lowering the leg rest /backrest.
18. Always check whether the surface of the massage chair is torn or ripped before use. If you find any unfamiliar tears in any area of the cloth in the backrest, do not use the massage chair. Kindly contact ITSU service center immediately for repairs.
19. This appliance is designed for household use only.
20. For those suffering from illnesses, please consult your doctor on whether you can use the massage chair if in doubt.
21. The following persons are requested to consult a doctor prior to use in order to reduce the risk of the user feeling sick or suffering from an injury:
  - Those who have a malignant tumor;
  - Those who suffer from heart disease;
  - Those who are experiencing pregnancy, menstruation and puerperal;
  - Those who require complete rest;
  - Those who are under medical treatment;
  - Those who suffer from back problems or whose backbone is abnormal or bent.
22. If you experience pain during use, stop the massage chair immediately. If any injuries happen, consult a doctor immediately. Never massage on swollen or inflamed parts of the body.
23. This product is a non-professional appliance designed to provide a comfortable massage. Do not use this product as a substitute for medical treatment.
24. Do not use this massage chair for more than 30 minutes at one time and kindly limit the massage to a concentrated area for not more than 15 minutes. Over-massaging can result in overheating, and thus reducing the product's lifespan.
25. Do not operate where aerosol (spray) products are used or where oxygen is being administered.
26. The massage chair is not suitable for children aged 12 years old and below.


# Safety Instructions

27. If the power plug is damaged, it must be replaced by a special plug available from ITSU or authorized agents.
28. As a safety precaution, Motor will stop working automatically (for protection) if apply too strong external force during use. Once appear above, switch OFF power, and switch ON again after 10 seconds. Please pay attention to this, especially for user whose weight over 100KG.
29. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
30. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved

## Warranty Claims

1. Warranty is not transferable.
2. This warranty is effective only if the product is purchased from ITSU.
3. Proof of purchase (original receipt) is required for all warranty repairs.
4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited to the duration stated in the warranty policy.
5. Damage due to misuse, improper treatment and unauthorized modification and repair are not covered by this warranty.
6. Warranty does not cover accessories and attachments which do not belong to the massage unit.
7. Warranty is not effective to rental, business, commercial, institutional or other non-residential users.
8. All services covered by this warranty must be approved by ITSU and repaired by authorized technicians only.
9. If replacement parts for defective materials are not available, ITSU reserves the right to make substitution in lieu of repair or replacement.


## Product Parts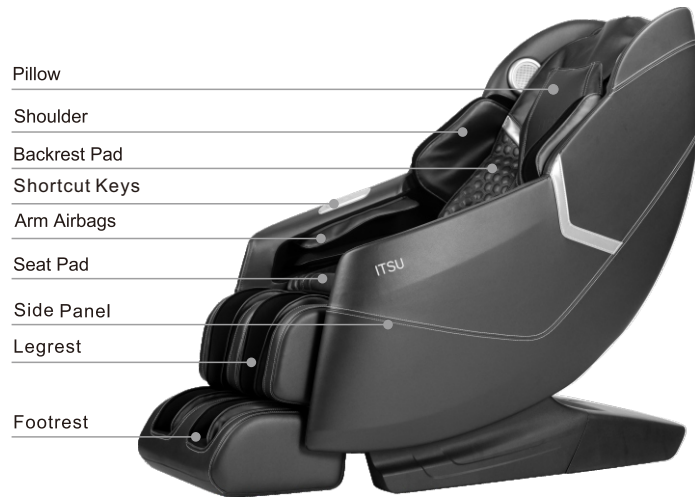

- Total 5 air valves ( 5 individual massage mechanism )
- A combination of 24 airbags

## Moving the Chair

As illustrated, first push the backrest downwards, and then tilt the massage chair onto its wheels to approximate 45 degree to easily move the chair around the house a light pull or push.

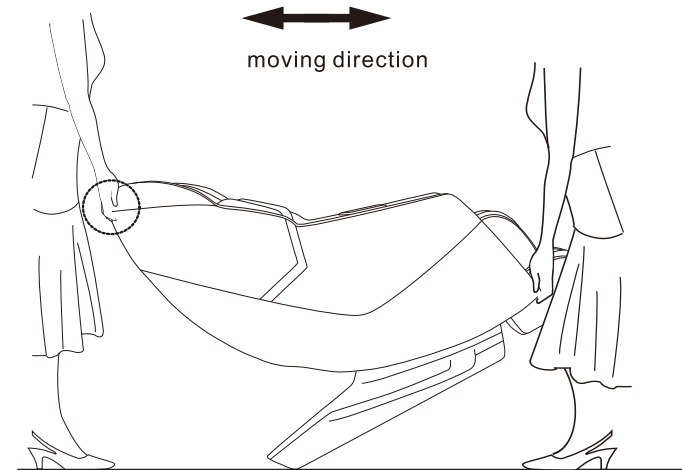
### Caution:

1. Do not move the massage chair when it is occupied.
2. Unplug the massage chair cable from power source.
3. Do not drop or suddenly let go of the chair when moving as it may damage the chair.
4. Two person participation is recommended.
5. Placing a mat underneath the massage chair is advisable, as moving process might induce scratches on the floor.


# Controller Diagram

## 1.Shortcut key function chart

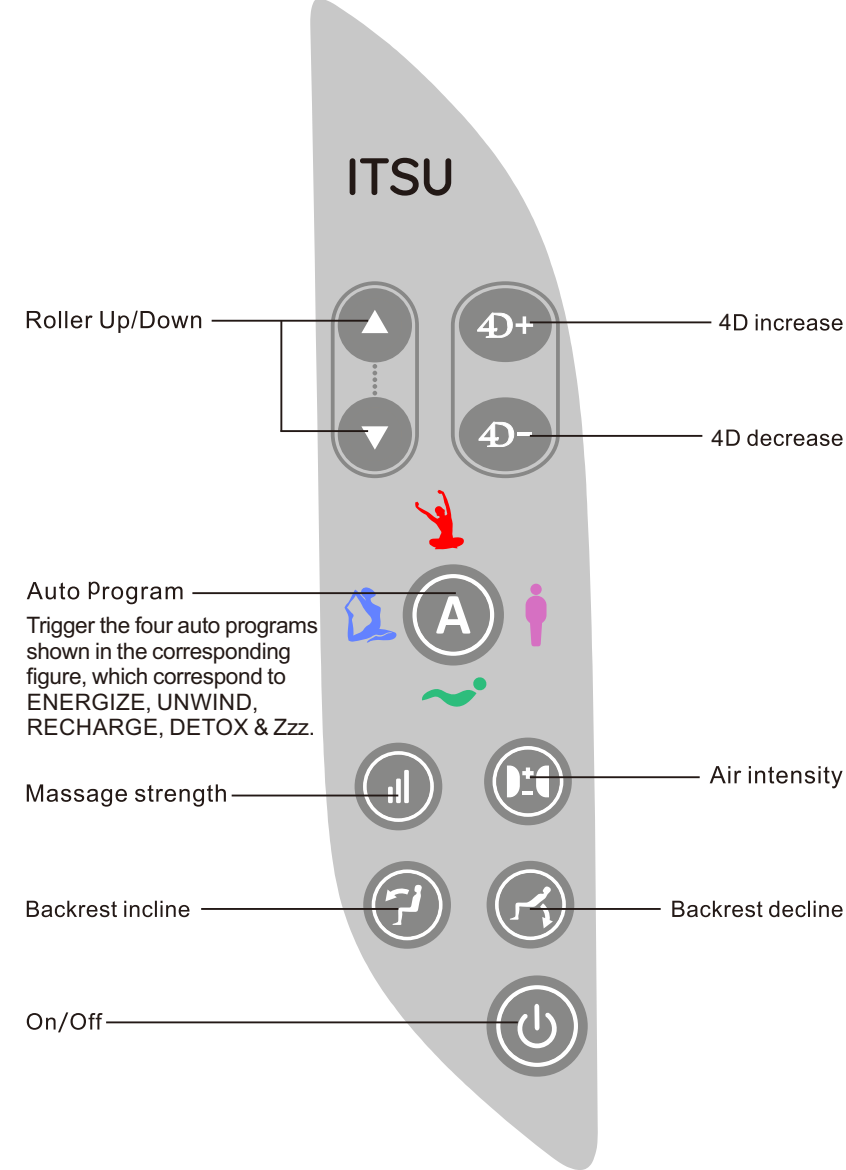# Controller Diagram

## 2.Remote Controller

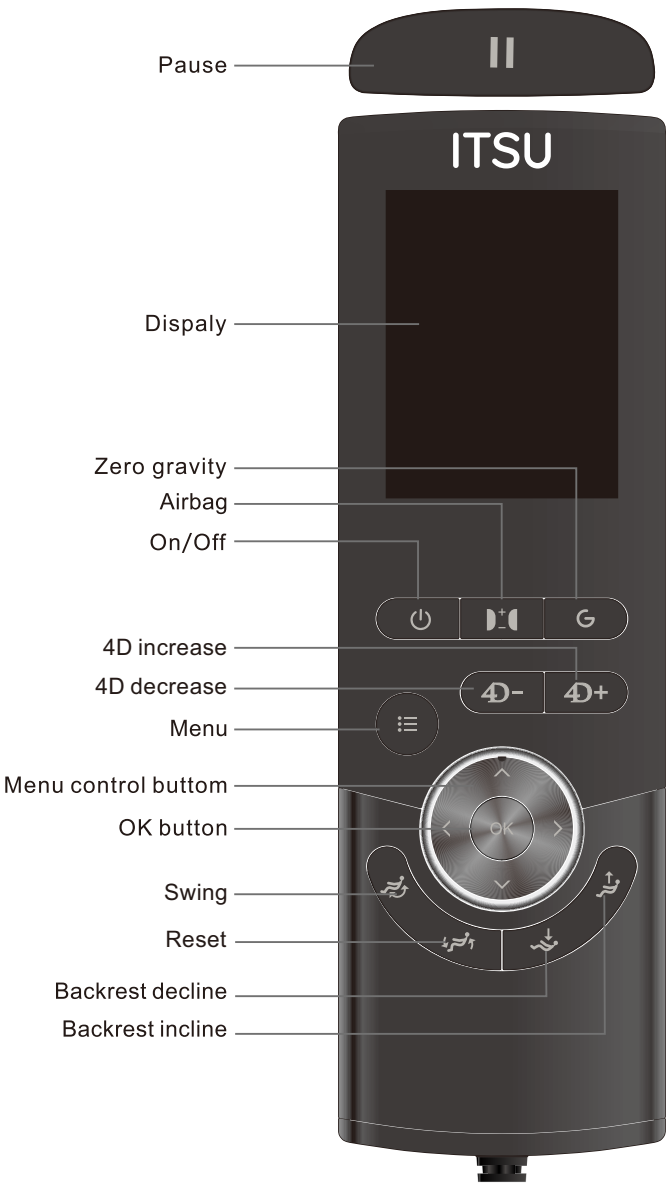# Product Operation Guide

## Function description of Controller buttons.

Press to PAUSE the massage chair .  
Press again to resume.

Press to switch ON the massage chair in default position.  
Press again to switch OFF.


Press to adjust airbag massage intensity.


## Position adjustment area:

Press to go to ZeroG position I .  
Press again to go to ZeroG position II .  
Press again to go to ZeroG positionIII.  
Press again to go to default position.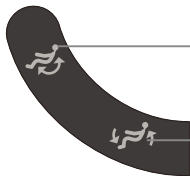

Press to enable SWING function, massage chair will begin to swing at preset interval.

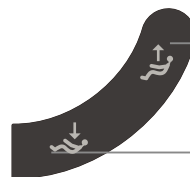Press to go to initial state.

Press and hold this button to adjust the backrest upward.

Press and hold this button to adjust the backrest downward.


# Product Operation Guide


Press to adjust 4D level. There are 6 levels.

Press to enter/return to Menu.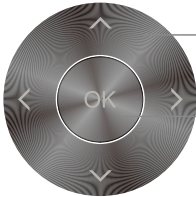


In Menu page, press these arrow button to nagivate to the respective function and settings.  
Press OK to confirm the selected function or settings.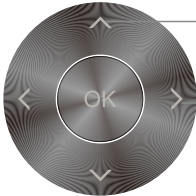

In Manual massage modes:  
Press UP/DOWN button to adjust massage roller upwards/downwards.  
Press LEFT/RIGHT button to adjust massage speed.


## Interface of remote controller and descriptions

1. Switch ON the massage chair, MENU will be displayed. Please select your desired program and function using ARROW KEYS. Press OK to confirm.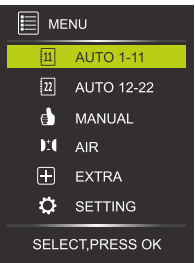


# Product Operation Guide

## 2.AUTO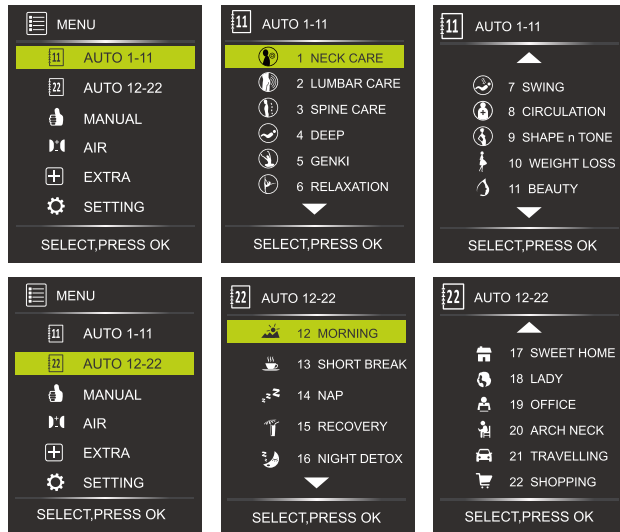

- There are 22 specially designed automatic programs as described above.  
 AUTO program default timer:  
 30 minutes program:Nap  
 15minutes program:All other program


### 3.Select your desired AUTO program, press OK button to confirm.

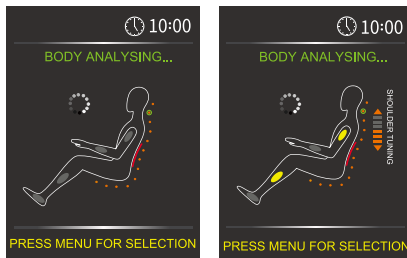- Massage chair will start with Smart Scan Body Sensing technology to detect body condition.
- After Smart Scan, in 8 seconds, users are given an option to fine tune 'shoulder position ' based on user needs.


# Product Operation Guide

## 4.MANUAL-MODE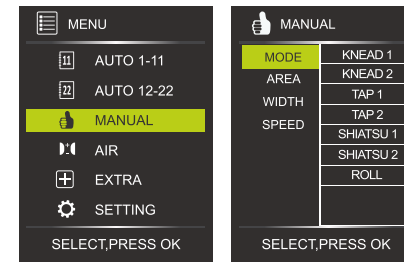


- 7 manual massage mode: Kneading1,Kneading2,Tapping1,Tapping2,Shiatsu1, Shiatsu2,Rolling.

## 5.MANUAL-PART

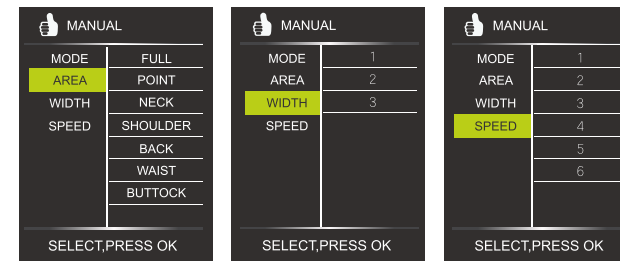- 7 manual massage coverage(part):Full,Point,Neck,Shoulder,Back,Wasit,Buttock.
- 3 width setting.
- 6 levels of Massage Speed/Strength setting.


## 6.AIRBAG

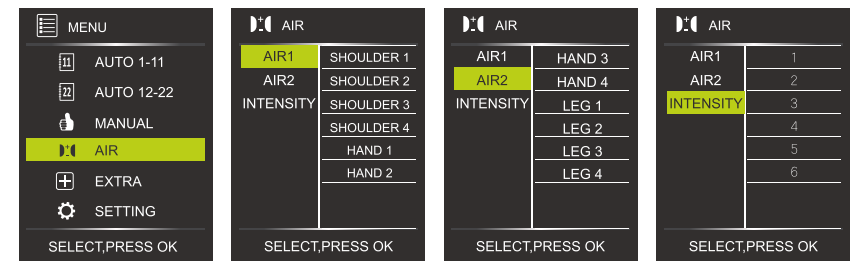- 12 airbag massage coverage(PART): Shoudler1,Shoudler2,Shoudler3,Shoudler4, Hand1,Hand2,Hand3,Hand4,Leg1,Leg2,Leg3,Leg4.
- 6 levels of air intensity, default airbag intensity is level 3.


# Product Operation Guide

## 7.EXTRA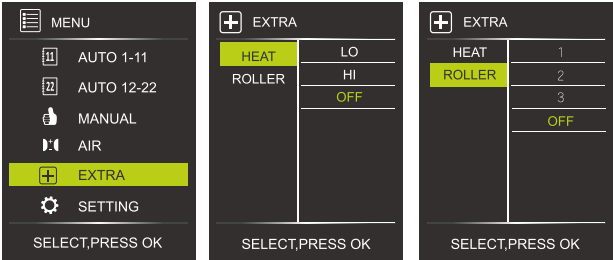

- 2 levels of heat.
- 3 modes of sole roller.


## 8.SETTING

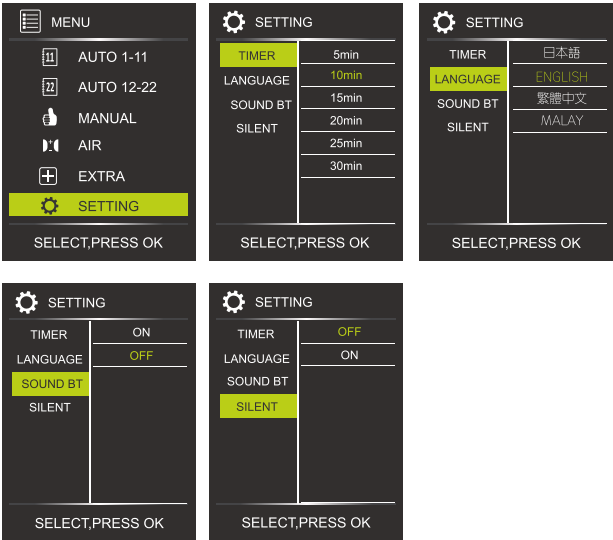- Timer setting at 5,10,15, 20,25 and 30 minutes.
- 4 languages are available to choose: Japanese,English,Chinese and Malay.
- Music bluetooth : enable/disable.
- Silent mode: enable/disable.

# Product Operation Guide

## Controller display

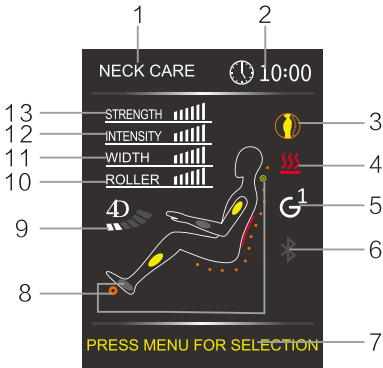- 1.Auto program name
- 2.Elapse timer
- 3.Massage technique
- 4.Heater
- 5.ZeroG activated
- 6.Bluetooth
- 7.Messages
- 8.Airbag position and roller position indicator
- 9.4D adjust
- 10.Foot roller strength/speed
- 11.Massage roller width
- 12.Airbag intensity
- 13.Massage strength

# Product Operation Guide

## APP Installation

1. Compatible with the following mobile operating system:

IOS 13

Android 10

Bluetooth 4.0

2. Turn on your mobile phone's bluetooth and WIFI or 3G/4G.
3. Scan the QR below to install ITSU WORLD app.

Android

IOS



# Product Operation Guide

## Interface of APP and descriptions

Switch ON the remote controller, HOME screen page will be displayed. Please choose your desire features. An extra auxillary HOME screen is also available with comprehensive features.

## Body Analyzing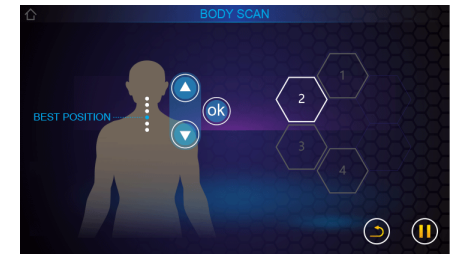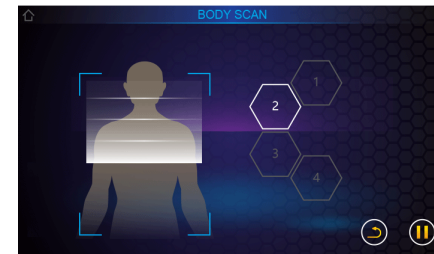

1. Body analyzing
2. Once completed, user are given an option (in 8 seconds) to fine tune 'shoulder position' according to user needs.

## AUTO

# Product Operation Guide

- There are 30 specially designed automatic programs as described above. AUTO programs default timer is 15 minutes except A14,A24,A30.

- Select SIGNATURE, to enter into 4 specially designed signature auto program.

Press to download/update new program. This function will be made available in the near future.

Warm back / back massage program, about 5 minutes and 30 seconds. Press the icon to switch ON/OFF.



## MANUAL

# Product Operation Guide

- 7 message modes: Kneading 1, Kneading2, Tapping 1, Tapping 2, Clapping 1, Clapping 2, and Rolling.
- 7 levels of Massage Speed/Strength setting.

## EXTRA

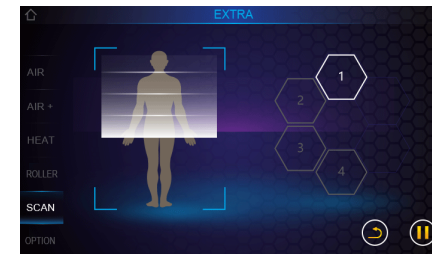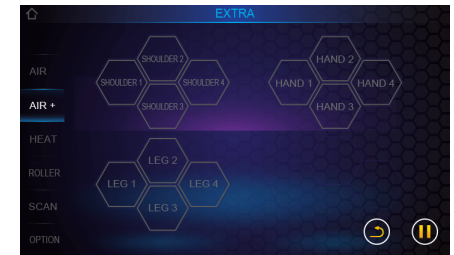- Additional features are group under EXTRA interface.

# Product Operation Guide


## SETTING

- 4 languages are available to choose: Chinese, English, Japanese and Malay.
- Screen brightness adjustment, 4 level.
- Remote controller volume adjustment, 4 level.
- Speaker volume adjustment, 4 level.
- Bluetooth music on / off.

# Product Operation Guide

Interface 1:

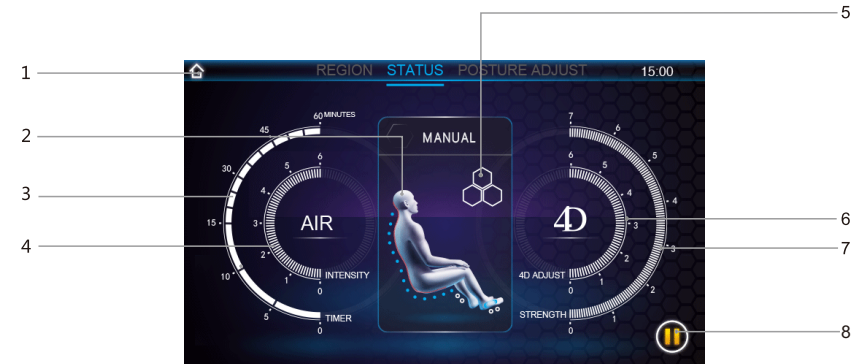1. Home button
2. Massage Indication
3. Timer setting
4. Air massage strength: 6 adjustable levels
5. Massage program shortcut button
6. 4D adjust: 6 adjustable levels
7. Massage strength: 7 adjustable levels
8. PLAY/PAUSE button

Interface 2:


1. Backrest incline/decline
2. Zero G position selection
3. Swing modes selection
4. Upright position

# Product Operation Guide

Interface 3:

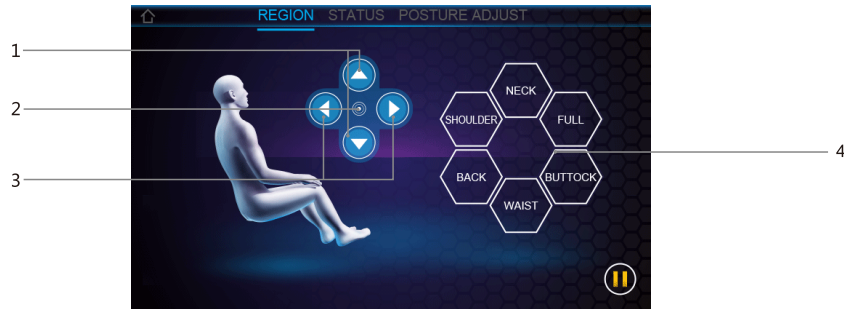1. Gear box roller upward/downward adjust
2. Spot massage function
3. Gear box width adjust
4. Massage area selection (for manual massage only).



Interface 4:

1. Movement strength adjustment
2. 4D adjust
3. Airbag intensity

# Product Operation Guide


## Wrist band sync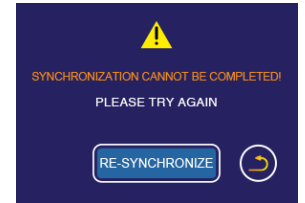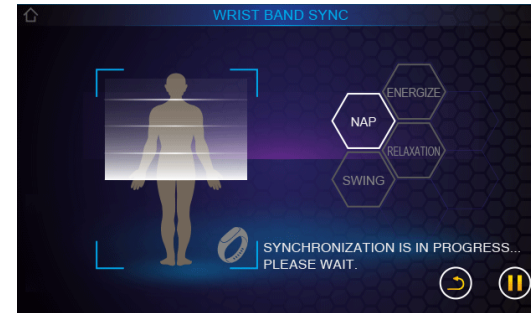

Click wrist band icon to begin data synchronization. Once completed, AI will recommend some programs. Please choose one program to start massage. In the event where synchronization is fail (maybe due to bluetooth not connected, no wrist band found or etc), a window will pop up. Kindly rectify the problem and re-synchronize again.


# Product Operation Guide

## After use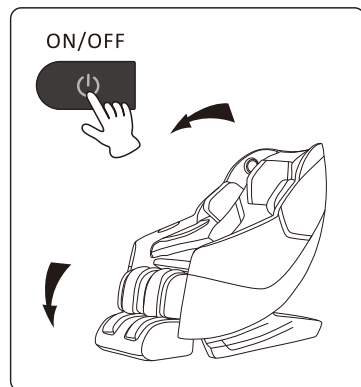

1. Switch OFF the massage chair to return back to original position.

- Please check surrounding, paying special attention to children or pets that may be located in close proximity, before adjusting massage chair backrest/footrest angle.
- Please make sure arms and legs are away from gaps to prevent any possible injury.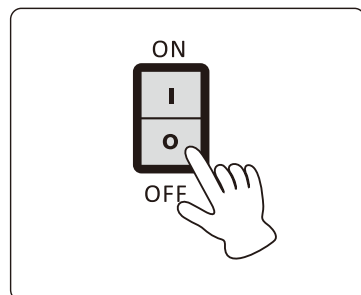

2. Press POWER button (located at the bottom back of the massage chair) to turn OFF the massage chair and restore the remote controller back to the storage compartment.

3. Turn OFF Main switch and unplug from power outlet.

- Do not pull the power cord to remove the plug.
- Do not unplug with wet hands.

# Product Features

ITSU massage Chair is controlled by a microcomputer that performs various intelligent massage methods and combinations, like Kneading, Tapping, Shiatsu, Clapping, Rolling, Footroller and Air Massage. All these massage methods accurately capture the acupressure points that could alleviate fatigue, promote metabolism and improve blood circulation effectively. Equipped with eight of full body automatic program, this product thoroughly fulfills your various needs.

Various types of manual / auto massage to suit your needs.

Spring-styled massage that is specially structured for the back is designed according to the curve of the human body to provide a more effective and thorough massage.


1. Designed according to body curve, to provide better massage sensations.
2. The Timer Function can be adjusted according to your desired duration of the massage programmer.
3. User friendly operation.
4. Built in with power surge and automatic temperature sensor for safety.
5. The operation is controlled by an intelligent microprocessor for excellent performance and consistent quality.
6. The machine radiates heat on the surface, Do Not use it over sensitive skin areas or in poorly-ventilated areas.

# Cleaning And Maintenance

## Cleaning of the synthetic leather

Wipe the synthetic leather with a soft dry cloth.

- **Do not clean the product with chemicals such as thinners, gasoline and alcohol.**
- When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.
- If the synthetic leather is very dirty, wipe it with the method below.
  1. Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
  2. Wipe surface of the synthetic leather with the soft cloth.
  3. Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
  4. Wipe with a piece of wrung soft cloth.
  5. Have the synthetic leather take natural air drying.
- If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it wipe the product.
- Do not dry the surface by a hair drier.
- Surface of the synthetic leather may be dyed by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.
- Keep these positions from long-time contact with plastics to avoid fading.
- Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.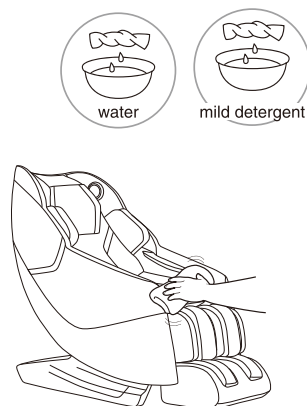

## Cleaning of the plastic parts


**1** Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.

**2** Wipe with a piece of cloth once soaked in clean water and wrung out.

- Wring the cloth out before using it to wipe the remote controller.

**3** Have the plastic parts take natural air drying.

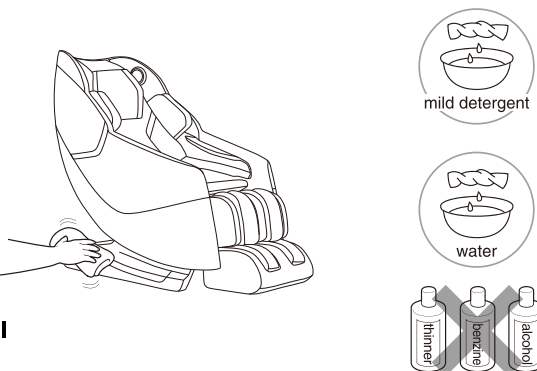# Questions and answers(Q&A)

**Q** Can persons paying frequent visits to hospitals because of diseases use the product?

**A** These persons should consult physicians before using the product. Massage is the so-called "touch-pressure stimulation": Muscles are compressed to relieve their tension and promote hemokinesis. Some diseases may get worsened because of massage. Thus, consult the attending doctors before using the product.

**Q** How about the body height requirement for use of the product?

**A** The body height requirement differs with the specific body type. See the rough reference below when using the product. If a person shorter than 145 centimeters or taller than 185 centimeters uses the upper body massage function and the Lower Leg/Foot massage function of the product at the same time, it will be possible that his soles will have poor contact with the foot massage section or that his knees will need to be bent.

For a taller person with knee bending when using the product:

Select method ②

② Tilt the backrest and lower the Lower Leg/Foot massage section.

For a short person difficult to touch the foot massage section:

Select method ③

③ Raise the backrest and lift the Lower Leg/Foot massage section.

**Q** Are there any things needing special attention during moving?

**A** Pay attention to the following things during moving.

For convenience during moving, tilt backrest of the product till the lowest position.

During handling, do not sit on the product.

① Press ON button power on the product.

② Press  to make the backrest completely tilt.

Press and hold on these two buttons until backrest and footrest stop.

③ Turn off the power switch and pull out the power plug.

● If the original packing box is unavailable, make sure the power line is not wound on the product, wind it and place it on the seat surface. When handling the product, keep the power line from falling off and rubbing with ground.

● If the original packing box is unavailable, properly keep other packing auxiliary materials and wrap the product with them to avoid scratch and vibration during handling.

**Q** I care about the sounds the product makes when being used.

**A** It is caused by the core structure.

**Q** Does it matter if the massage wheels have different massage intensities?

**A** This phenomenon possibly exists during massage because of the core structure.

To simulate the most natural massage effect, the product is designed with a core structure allowing alternate back massage by a left massage wheel and a right massage wheel. During massage, the massage wheels possibly have slightly different positions on the back. That's why you have that feeling. It is not a fault.

## Troubleshooting

**After a fault happens, inspect and try the corresponding method below.**

If the fault still exists after that, contact the shop from which you bought the product.


Fault	Cause and elimination method
The massage wheels stop during use	<p><b>For safety, the massage wheels will possibly stop upon a too large external force during use.</b></p> <p>At this point, you need to adjust your sitting position, and don't put your back completely on the backrest. (Pay attention to the relevant precautions, especially when used by customer whose weight is 100 Kg or more.)</p>
The backrest can not recline up and down or the footrest can not be lifted up and down	<p><b>For safety, the product will stop upon an obstruction or a too large force.</b></p> <p>If all contents and buttons are flickering and the backrest or the footrest has stopped, turn off the power supply and turn it on about 10 seconds later and then operate the product.</p>
The massage wheels cannot reach the shoulders or the neck	<p>If the head is not in contact with the headrest, or the back is not in contact with the backrest, the shoulder position in body type detection will possibly be lower than the actual position. Sit in the deepest position of the product, lean your head on the headrest and operate from the beginning again.</p>
The left and right massage wheels have different heights (intensities)	<p>It is not a fault but a normal phenomenon because the product is provided with a massage core allowing alternate beats of the massage wheels.</p>
The product cannot run at all	<ul style="list-style-type: none"> <li>● The power line is disconnected.</li> <li>● The power switch is not turned on.</li> <li>● No program is selected after  on the remote controller is pressed.</li> </ul>
No warm feeling in backrest	<ul style="list-style-type: none"> <li>● The heating button is set to be OFF.</li> <li>● The warm feeling depends on factors such as body type, massage method, clothes and room temperature. Please keep back close to the heat-sensing conduction part to make the customer feel warm.</li> </ul>
The power supply cannot be connected even if  is pressed again after the massage	<ul style="list-style-type: none"> <li>● The program will terminate about 10 seconds after the massage has ended. Press  at least 10 seconds after all the movements have stopped.</li> </ul>
The product is damaged	
The power line or the power plug has abnormal heating	<p>To avoid accidents, contact an authorized service center.</p>

## Troubleshooting

### The sounds and feelings you care about in use of the product

**You will hear or have the following sounds or feelings when using the product. It is not abnormal phenomena but results of the product structure.**

- Sound when the stretching massages are retracted
- Sound and feeling when the massage wheels rub patterns on the fabric
- Sound of movements such as kneading and beat
- Friction sound of the running massage wheels
- Sound of up-down movement
- Sound of the conveyor belt
- Motor sound
- Friction sound between the armrests and the back cushion when the backrest is tilting
- Friction sound between the massage wheels and the fabric
- Sound of the inflator pump during Air Bag massage
- Sound of air exhaust during Air Bag massage
- Running sound of the solenoid valve at bottom of the product during Air Bag massage.
- Swing of the footrest
- Running sound of the air bags
- Sliding sound of the foot massage section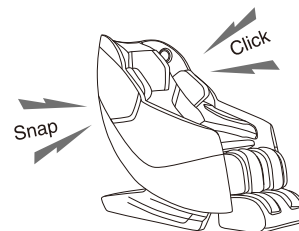

**If the sound is loud, it is also possible that the product has failed.**

- In this case, have the product inspected and repaired by an authorized service center.

### Warning


- **Do not refit, disassemble or repair the product without permission.** Otherwise fires, abnormal movements or personal injuries may happen.

### Warning

- **In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid fuming, fires and electric shocks.**
- < Abnormality and fault examples >
- The product fails to work after the power switch is turned to the ON position;
- After the power line is moved, the product is discontinuously powered on;
- In use, the product gives off a burning smell or an abnormal sound;
- The product is deformed or abnormally heating
- In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.


# Product Specifications

Power supply		AC220-240V~,50/60Hz
Power consumption		100W
Automatic power-off		Approx. 30 minutes
Dimension	When it is not tilting (L x W x H)	Approx. 1420mm x 740mm x 1170mm
	When it is tilting (L x W x H)	Approx. 1650mm x 740mm x 920mm
Weight		Approx. 65kg
Dimensions of packing box (L x W x H)		1590mm x 765mm x 875mm
Weight(including the packing box)		Approx. 78kg
Operating Conditions		Ambient temperature:Approx.0°C~35°C Relative humidity:Approx.20%~80%
Storage Conditions		Ambient temperature:Approx.-5°C~35°C Relative humidity:Approx.20%~80%
Maximum body weight of user		Approx. 120kg
Entrance size		Minimum 74cm

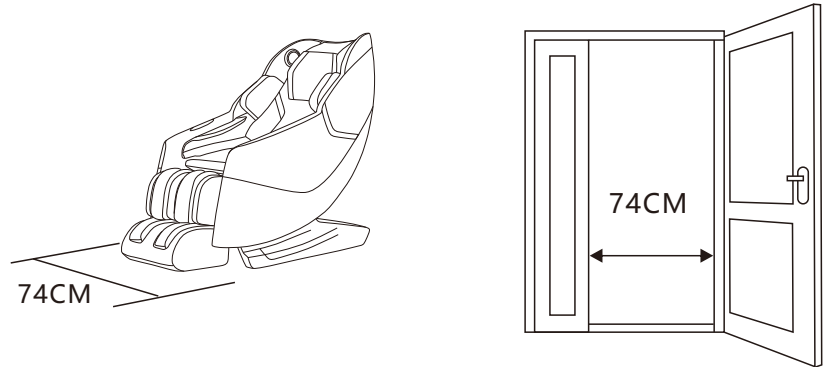For a user heavier than 100 kilograms, the product may have bigger noise and its fabric housing may be worn quicker.

# Product Specifications
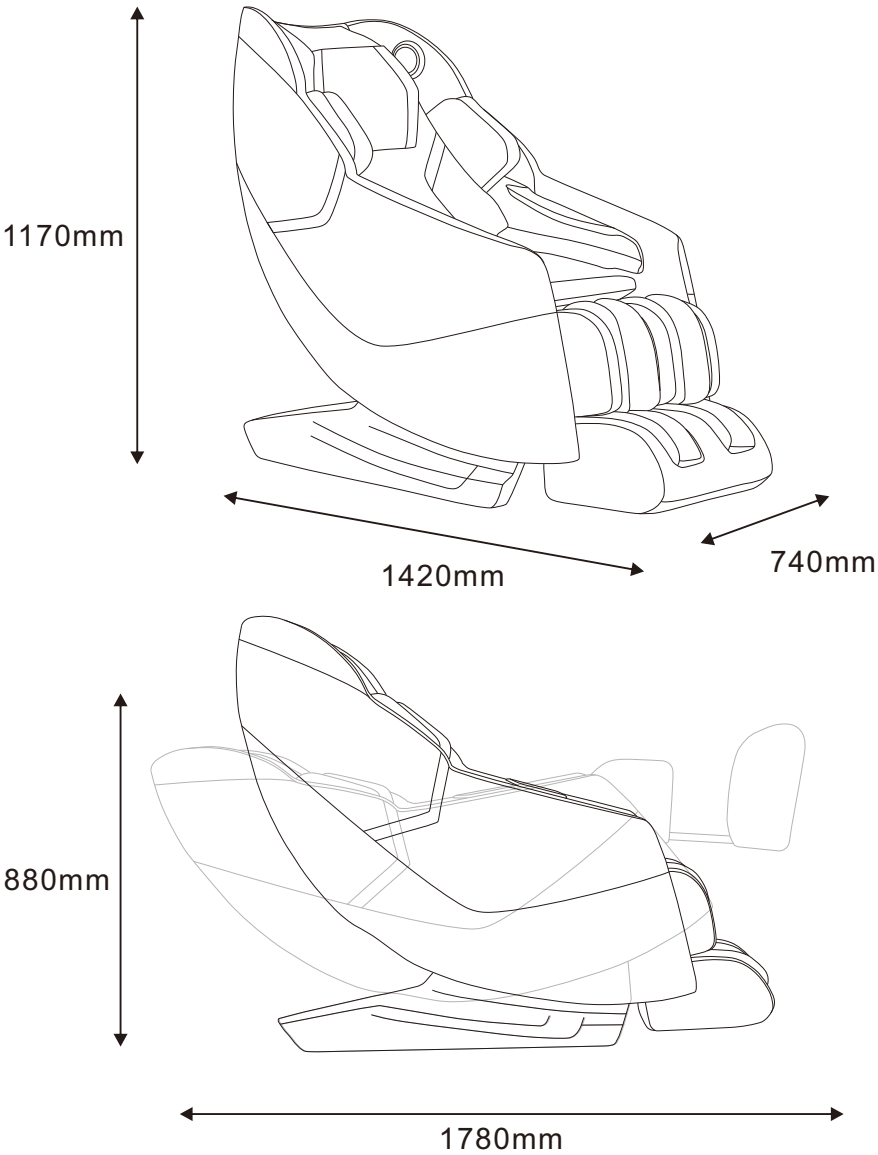
## Contact Us

ITSU World Sdn Bhd  
Unit BO03-B-13A-1,  
Menara 3A, KL Eco City,  
No.3, Jalan Bangsar,  
59200 Kuala Lumpur.  
Tel: 1700-81-4878

ITSU World(HK) Limited  
Flat A6, 20th Floor, TML Tower,  
No. 3 Hoi Shing Road, Tsuen Wan, N.T.  
Tel: +852 3611 2228

PT Healthy World  
Lodan Center Block O No 20  
Jalan Lodan Raya No 2  
Jakarta Utara 14430  
Tel: +62 21-6983 1123

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